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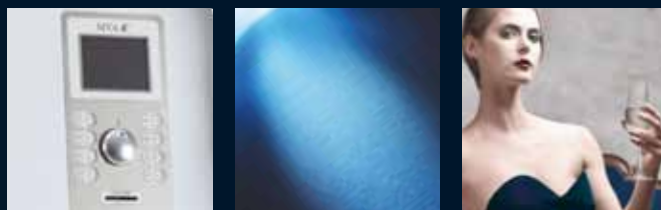
Certified Beauty - Cesare Quaranta Quality

Pushed by the passion of its founder and strong with its own craft Cesare Quaranta srl is a certify company for quality system managing according to standards UNI EN ISO 9001:2008 for design, technology production, and technical assistance on easthetic devices.

The top management decision to follow a System for Quality Management is due to the willing of respond to customers needs.

The quality system has the peculiarity to reorganize the company in order:

- To meet and to exceed, if possible, the expectations of the customer in terms of product quality;
- To consolidate and to increase market position building customer loyalty
- To stabilize and to improve the production process in order to ensure the best Functionality and reliability
- To pay attention to cost containment.

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THE BEAUTY CULT . ART IN SLIMMING

Mya E+ is a device studied for whom want to shape the silhouette and to whom want to became expertise in the business of slimming and treating of localized cellulite.

Mya E+, a Cesare Quaranta patent, borns thanks to scientific research carried out during the years and to millions of beauty treatments provided in worldwide institutes. The real innovation of Mya e+ is a professional and scientific approach in slimming and contrasting cellulite throughout:

- Powerful pads emitting US and Electrostimulation
- New beauty protocols
- A total body check up

Body treatments:

- Streamlining
- Drainage
- Shaping
- Firming
- Cellulite

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"TOTAL BODY" WORKSTATION.

Mya E+ is a total body workstation that assures more performing body treatments and innovative visage methods. It simultaneously works on face and body and, same as previous generation, it has been designed for working on total autonomy without the support of the operator during the treatment.

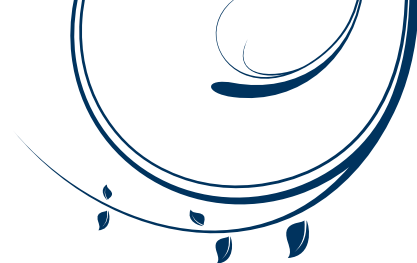
For body:

- powerfull and improved ultrasound plates for slimming and cellulite contrasting treatments even more effective, rapid and long-lasting;
- Wave forms designed to enhance the results in oxygenation and regeneration of tissues, helping exchanges and metabolic elimination of toxins;
- Electroporator handpiece with 4 colors chrome stimulation leds that transfers to the tissues "active cocktails" for slimming and for fighting the imperfections of cellulite;

For Visage:

- A red "LED anti-aging masque". It stimulates the photoreceptors biological response and it is really effective in contrasting the signs of aging and sagging skin.
- A electroporator handpiece that vehiculates the active ingredients on tissues contrasting wrinkles, rehydrating tissues and favoring skin lifting effect.

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BODY CHECK UP IMPERFECTION SCIENTIFIC APPROACH.

The first point for programming a proper plan of slimming treatments is to identify the somatotype and the morphology of the subject to be treated. Different subjects respond - to equal treatment and feeding - in a different way and do not reach same results.

During the first interview with customer, and periodically during the course of treatments, it will be measured the change in body mass, monitoring constantly the results achieved

through a precise body check up in 6 steps:

- Measurement of weight and height.
- Evaluation of body mass through multifrequency bio-impedance test
- Centimeter evaluation
- Plicometry
- Photographic imperfection detection
- Thermographic analysis for the measurement of cellulite* stage

The software processes all information through complex algorithm. It exactly identifies the morphological person profile and the body mass percentage, fat mass. Intra and extra cellular fluid in the body. The system indicates with precision the metabolic response to slimming program.

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BIO IMPEDANCE TEST WITH THREE FREQUENCIES CURRENT

Thanks to the bio-impedentiometric test with which Mya is equipped it is possible to evaluate the client's body composition, by monitoring the condition before undertaking the treatment and by observing the improvements during all the session cycle.

The test allows carrying out, thanks to the action of a three-frequency sinusoidal current, precise and reliable measurements about the percentages of thin mass, fat mass, total intracellular and extracellular liquids of the subject.

Carried out before a session cycle, it allows elaborating a personalized work protocol aimed to reach the aimed aesthetic targets in the shortest possible time. After the test,

the software with which Mya is equipped suggests the number of sessions and the ideal type of treatment for each subject.

Each one of the three frequencies used by the software acts like a fundamental function in the subject screening:

- **5 kHz:** contributes to evaluate the extracellular water;
- **50 kHz** is used for the thin mass measurement;
- **100 kHz:** allows the total water measurement.

The test is advised particularly for all the treatments aimed to the fat mass reduction, such as the localised fats and to fight the imperfection of cellulite sold out of the product contained in it.

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US AND ELECTROSTIMULATION

Mya E+ combines the benefits of ultrasound and electrostimulation on same time on the same support. It guarantees immediate and lasting results in slimming, shaping and cellulite imperfection treatments

The **ultrasound** combines localized thermal and micromechanical effects, which make cellulite tissue component structures weak. In particular the adipocyte membranes. Ultrasounds improve vascularisation, tissue massage, local heat production and cause molecular dissociation which leads to cellulite nodule breakup.

The **electrostimulation** act on face and body thank to deep impulses. It improves muscular mass increase, tissue oxygenation and lifting effect. It stimulates the lymphatic-venous circulation, thus providing slimming and lipolytic effects and it increases the body general wellbeing. Controlled currents emissions with different wave forms are modulated at frequencies ranging from 33 Hz to 5.000 Hz to allow for performing personalized and effective programs.

Mya E+ features 28 programs. Each program can be personalized easily. It is possible to obtain different results on the same stimulated area in just one session.

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4 LED CHROMO STIMULATION AND BODY ELECTROPORATION HANDPIECE.

The handpiece focuses the action of chromo stimulation to four colors led and it combines the effectiveness of electroporation in delivering trans dermal active ingredients.

Handpiece treats the imperfections uniformly distributing the cosmetic on the skin and favoring the total absorption of active ingredients dissolved in it through electroporation action. It has the task of inducing small reversible structural changes in the stratum corneum, in this way it improves the permeability of tissues.

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ANTI AGE LED MASQUE

The face mask with red LED stimulates the biological response of the photoreceptors by inducing an effective biochemical reactions of tissues contrasting the signs of aging and sagging skin. The stimulation is athermic, painless, and it is suitable for any skin type. Benefits are:

- Increased cell turnover in the epidermis;
- Increased production of collagen by fibroblasts;
- Improvement of skin texture;
- Reduction of fine lines;
- Firming tissue;

The bio-stimulant action is exerted through the "LED anti-aging masque" which, thanks to 70 LED sources, distributes equally intensity and dose on the face surface. The result is a more toned, supple, youthful and healthy complexion in just few sessions.

activa[®] MASK ANTI AGE

is an intensive treatment with polynucleotides for for the aesthetic recovery of the skin in order to counteract face and neck skin aging.

Nucleotides are natural substances that enter the structure of nucleic acids (DNA), taking part to the molecular processes of coding and transferring genetic information to organisms cells.

The special formulation of **activa[®]** Anti Age Mask - consisting of peptides with low molecular weight - is specifically designed to facilitate the absorption of active ingredients by the skin, increasing the aesthetic results throughout LED irradiation.

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Standard Equipment

N° 1 NYLON CONTAINER WHICH INCLUDES:

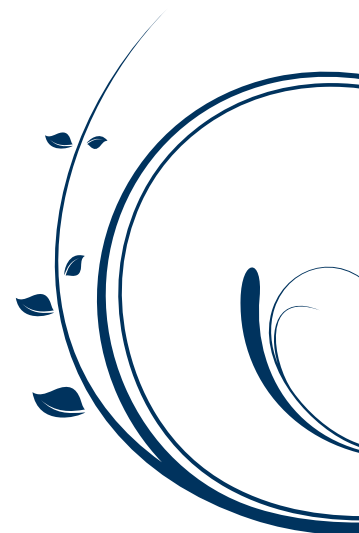
- o N° 16 ULTRASOUNDS STIMULATION PADS
- o N° 2 JUNCTION BOXES
- o N° 2 25-PIN CABLES, 2+3 m.
- o N° 2 8-PIN CABLES
- N° 1 EMERGENCY REMOTE STOP SWITCH
- N° 1 4-PIN CABLE FOR IMPEDANCEMETRY
- N° 32 STICKY AND CONDUCTIVE ELECTRODES, PG 900
- N° 32 STICKY AND CONDUCTIVE ELECTRODES, PG 1000
- N° 16 STICKY AND CONDUCTIVE ELECTRODES, PG 1500
- N° 2 OVAL ELECTRODES for BIA, WITH BAND INCLUDED
- N° 16 SPLITTERS
- N° 16 CLIPPED ADAPTERS
- N° 10 CHIP CARDS
- N° 1 POWER MAINS CORD (3 m long)
- N° 4 40cm ELASTIC WRAPS
- N° 6 60 cm ELASTIC WRAPS
- N° 4 80 cm ELASTIC WRAPS
- N° 2 100 cm ELASTIC WRAPS
- N° 1 PACKAGE OF ULTRASOUND GEL, 260 g
- N° 1 ANTI AGE LED MASK
- N° 6 BOTTLES ACTIVA40 SNELLY GEL
- N° 6 BOTTLES ACTIVA40 GEL COADJUTANT IN THE TREATMENT OF THE CELLULITE
- N° 1 INSTRUCTION MANUAL
- N° 1 METHODS OF USE

Activa⁴⁰ SNELLY GEL

Activa⁴⁰ Snelly Gel is the new cosmetic gel to be used with Mya E+

Activa⁴⁰ Snelly Gel carries out a double action: mechanical and cosmetic. It allows the ultrasound passage, carrying out at the same time a strong slimming effect. It is rich in active ingredients, ruscus and escine, promoting the reduction of oedema, the liquid drainage and an anti-inflammatory effect.

The result is a highly functional, cosmetically pleasant and user-safe product.

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MADE IN ITALY PASSION

Cesare Quaranta Technologies is the expression of a passionate company which, for over than 25 years, designs and manufactures its equipments in Italy.

For Cesare Quaranta and his staff "Made in Italy" means ability to transform its own exclusive know how in reliable, flexible and innovative equipments responding to the real needs of aesthetic operators: safe and effective treatments. customer.

Cesare Quaranta production is cared in every detail, constantly able to innovate and to transform the most complicated hi-tech solution in a friendly user technology.

ASSISTANCE AND WARRANTY

Certainty to have specialized technicians at your complete disposal at any moment.

800/26.86.16 is the toll free number for the Client assistance. Cesare Quaranta is at your disposal to supply you with immediate services of technical intervention, and to answer all your requests or to get back to you with the information regarding the use of the devices.

Cesare Quaranta devices are guaranteed against defects in material and construction for a 12-month period.

Cesare Quaranta s.r.l. is synonym of high quality service towards any centre offering aesthetic and wellness treatments: the devices distinguish themselves by their compliance with the highest quality standards and by the extreme efficacy of their own methods.

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Power Supply: 220/250 V. single phase 50/60 Hz
Absorbed Power: 170 VA MAX

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ELECTROSTIMULATION

Output Stage
 Square wave - Kotz current
Square Wave Frequency
 1kHz modulated at 33/66/133/200 Hz
Kotz Wave Frequency
 2500/4000/5000 Hz modulated at: 4/6/8/12/50/75/100 Hz
OUTPUT CURRENT
 80 mA on a 500 Ω load

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ULTRASOUND

Ultrasound Frequency: 3 MHz
Ultrasound Power: < 3 W/cm²

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COSMETIC ABSORPTION AND COLOURS

Output Stage: square wave
Square Wave Frequency: 1,6 kHz modulated at 300 mHz
Max Output Current: 80 mA, load 500 Ω
Colours: Blue, white, yellow/orange, green

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HELMET

Led: Visible red light
Dimensions: 300 x 300 x 380
Led Number: ~70
Wave Length: 645 nm
Absorbed Power: 4 W
Number Of Output Channels:
 8 right channels and 8 left channels for ultrasound, 4 right channels and 4 left channels for electrostimulation; they can be adjusted individually for each output, with analogue and digital indication on the display
Treatment Selection:
 13 preset programs for traditional electrostimulation
 13 preset programs for ultrasounds and/or electrostimulation
 1 electroporation program
 1 impedancemetry program
 1 chip card program
 1 counter program
 1 settings program
Treatment Total Time:
 Preset or to be set, from 5 to 60 minute
Main Fuses: 2 x 2 A type T 5 x 20
Size: 420 x 1330 x 470 mm
Weight: 22 kg
Temperature Limits: 5° C to 40° C (when ON)
Classification:
 Equipment for Aesthetics
 CLASS I, BF TYPE
Standard Compliance:
 2004/108/EC DIRECTIVE
 2006/95/EC DIRECTIVE

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