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Active Press®

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Naturally light

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THE PRESS-MASSAGE
that stimulates circulation
and defeat unsightliness
of cellulite.

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From the first session you will feel
your legs, ankles, feet and arms
naturally light.

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Stimulation of the lymph-venous circulation

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What is Active Press

Active Press® is a digital equipment, able to set out the proper pressure required by different sectors of the jamb, ensuring the **maximum safety and effectiveness in each phase of the treatment**. The performed press-massage is:

- **SAFE**, with a system of automatic **pressure setting, ideal for each sector**.
- **Effective since the first session**, helps to defeat unsightliness of cellulite*, water retention, shaping of the silhouette.
- **Versatile as it offers a wide range of treatments** by means of memorized programs, which can be further on customized.

How does it work

Active Press® is an essential equipment for professional beauty centers. It carries out a controlled pressure which leads to a **lymph-venous drainage, extremely effective to defeat any beauty defects such as cellulite*, water retention, oedemas, heaviness of lower and upper limbs**.

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- **Linfo-drain:** it performs a lymphatic drainage, according to Vodder method for who have remarkable liquids stasis, tissues trophism and cellulite*, resulting from a bad lymphatic circulation.
- **Revitalizing:** a sequence of pressures which act to a more superficial level. Particularly indicated to stimulate and to improve blood circulation; it performs a revitalizing action which brings more oxygen to tissues and helps slags elimination.
- **Total:** it is based on contemporary pressure of all sectors and, afterwards, their contemporary emptying.
- **Physio:** it performs a massage of the connective tissue, whose effect brings positive consequences to deeper tissues. This program renews and cleans liquids, leading to a regenerating action on tissues.
- **Total physio:** similar to program 4, with the only difference that the emptying starts from the ankle, keeping a more lasting draining effect. It is indicated for who suffers important lymph-venous problems such as wearying of blood vessels, elephantiasis, extensive oedemas, lymphatic insufficiency.
- **Mix cell:** it is the ideal program to defeat unsightliness of oedemous and fibrous cellulite*. The press-massage carries out a deep draining action, leading to important benefits for all lymph-venous system.
- **Mix relax:** it is based on the pressure of each sector, one by one. The pressure is always carried out starting from bottom to the top of the body. It is recommended in case of venous oedemas combined with medium-low problems of superficial circulation; very good to mitigate legs heaviness.
- **Normal:** it is specially indicated for mixed cellulite* combined with localized adiposity and light circulatory disorder.
- **Personal:** it makes a sequential pressure of the different sectors. The shift from a sector to another is gradual, by pumping and emptying each sector, before passing to the next one. All data inserted by the operator will be automatically stored by the equipment.
- **Arms:** it is a pressotherapy of a medium-high range from shoulder to hand.
Arms kit is needed (optional)

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Did you know that...

The **pressomassage** appeared in the aesthetic world about 15 years ago. It is a methodic, that since its debut, has aroused great interest and obtained lots of success thanks to its easiness, to the results obtained and to the immediate comfort sensation it transmits. **The treatment consists in a physical action on the venous and lymphatic circulation by making it more effective and improving consequently inaestheticisms such as cellulite*, liquid retention, aedemas, loker limb heaviness, etc.** Since then, the technological evolutions have been numerous, but the best equipment are those that can exert dosed pressures to the tissues, with a pressure regulation adaptable to the needs of the person in treatment.

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EQUIPMENT ACTIVE PRESS®
PAIR OF JAMB BLADDERS
ARMS KIT (OPTIONAL)
PAIR OF PLANTAR BLADDERS
PAIR OF INGUINAL PRESSING BLADDER
ABDOMINAL BLADDER CORSET
PAIR OF ARMS BLADDER (OPTIONAL, UPON REQUEST)
STOP REMOTE CABLE
POWER SUPPLY CABLE
MANUAL OF USE AND INSTRUCTION

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8

Treatments selection

9 +1 pre-set programs, 1 of them is to be personalized and self memorizing

Main fuses

2 x 1,6 A type T 5 x 20

Dimensions

400 x 370 x 390 mm

Weight

7 Kg.

Temperature limits while using

from 5° C to 55° C

Device ranking

Aesthetics Professional Equipment, class I TYPE BF

Laws compliance

2004/108/CE, 2006/95/CE
CEI 62-39

**ENVIRONMENTAL CONDITIONS
FOR TRANSPORT AND STORAGE:****Temperature**

from -15°C to 55° C

Relative humidity

from 20% to 90% not under condensation

Pressure

600 -1060 hPa

Copyright Cesare Quaranta Srl 2008. The rights of translation, electronic records, copy, total or partial adjustment by whichever instrument (microfilm and photostatic copies included), are reserved . The copyright is enforced in all Countries.

CESARE QUARANTA s.r.l. Unipersonale
Via Legnanino, 5 - 10023 Chieri (TO) Italy
Tel. +39 0119427967 - +39 0119472132 Fax +39 0119472299
www.cesarequaranta.it - e-mail: info@cesarequaranta.it

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